


NEW

# DISPO VAN PEN NEEDLE




*Precision  
Engineered,  
Affordable*


## How to use **DISPO VAN** Pen Needles?

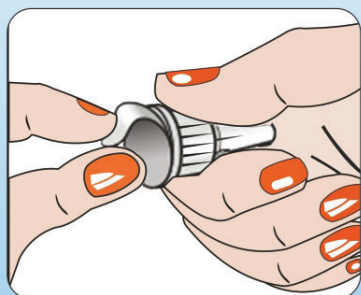


**Step 1:**  
Take a new needle. Peel off the paper tab

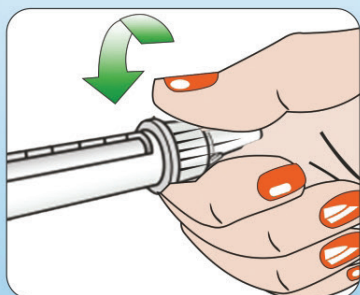

**Step 2:**  
Push the needle straight onto the pen.  
Rotate until it is on tight.

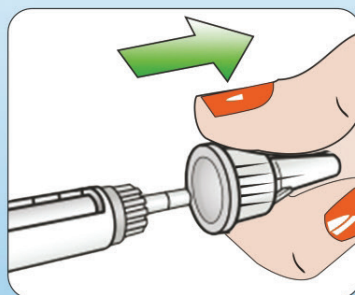

**Step 3:**  
Pull off the outer needle cap and keep it for  
later. You will need it after the injection to  
safely remove the needle from the pen.

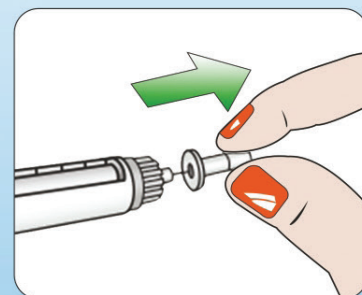

**Step 4:**  
Pull off the inner needle cap and throw it  
away. If you try to put it back on, you may  
accidentally stick yourself with the needle. A  
drop of the medicine may appear at the  
needle tip. This is normal, but you must still