




dispojektTM

Safety Needle with Sharps Injury Prevention
SIP SHIELDTM

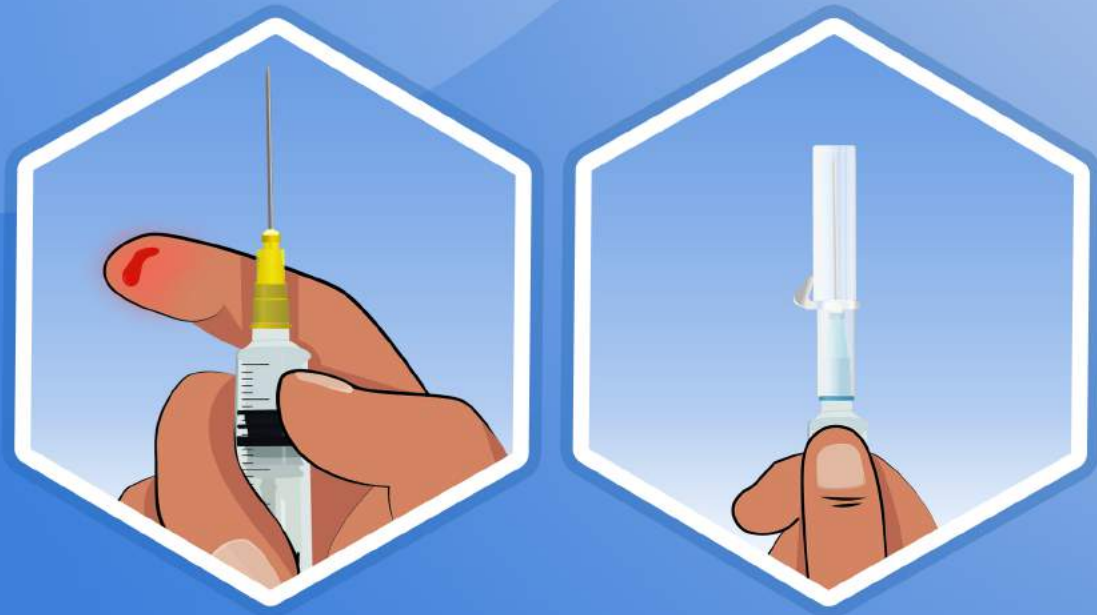

Safety from Accidental Needle Injuries
taking care of those who take
care of us



Features of dispojekt Sip Shield

- With safety cover
- Transparent safety mechanism activated by one hand.
- Audible click sound confirms locking of safety device
- Does not require a change in injection technique
- Sharp three –beveled needle to minimize discomfort
- Siliconized needle minimized penetration and gliding force
- Colour coded hub to conform to ISO 6009
- Available in 22G, 23G and 24G.
- Available in thin wall regular level.

Intended use

Used with syringes for single use hypodermic / intravascular / intramuscular injection.

Packaging details

The Dispojekt single use Safety Needle with SIP (Sharps Injury Prevention) are individually packed in transparent thermoformed peel able blister pack as a strip of five pieces.

Product range

Description	Gauge (G)	Length (mm)	Colour
Single use Safety Needle with Sharps Injury Prevention 22G x 1" (0.7x 25 mm) TW regular bevel	22G	25 mm	Black
Single use Safety Needle with Sharps Injury Prevention 23G x 1" (0.6x 25 mm) TW regular bevel	23G	25 mm	Blue
Single use Safety Needle with Sharps Injury Prevention 24G x 1" (0.5x 25 mm) TW regular bevel	24G	25 mm	Purple

As per ISO standard

Caution

- Reuse of product may lead to cross contamination / infection.
- Store in clean, dry & insect free place.
- Avoid exposure to acidic / corrosive fumes to avoid corrosion /rust.

Warning

- Do not use if the pack is previously opened or damaged.
- Do not attempt to re- sterilize and reuse.

Storage conditions:

- Store in clean, dry & insect free place.
- Avoid exposure to acidic / corrosive fumes to avoid corrosion /rust.






Step 1

Peel opens the blister pack and take out the Dispojekt safety needle.



Step 2

Push the safety needle onto a standard luer slip connector.

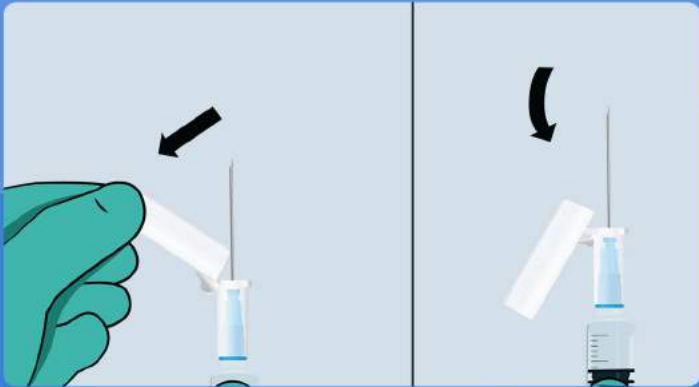


Step 3

Pull back the hinged cover to expose the needle.




Step 4

Fill the syringe with medication.

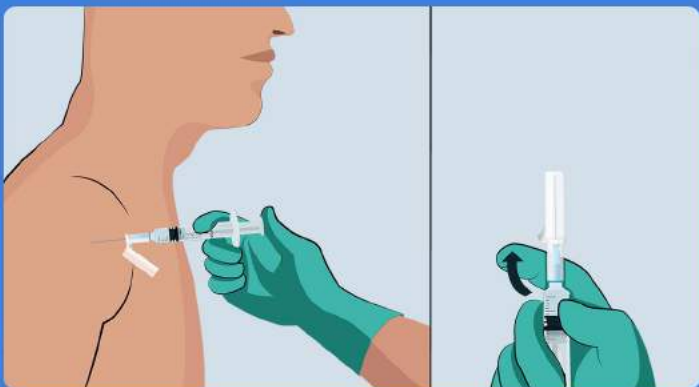

Step 5

Orient the needle to enable bevel to face upwards. Administer the injection and close the needle cover immediately after use.

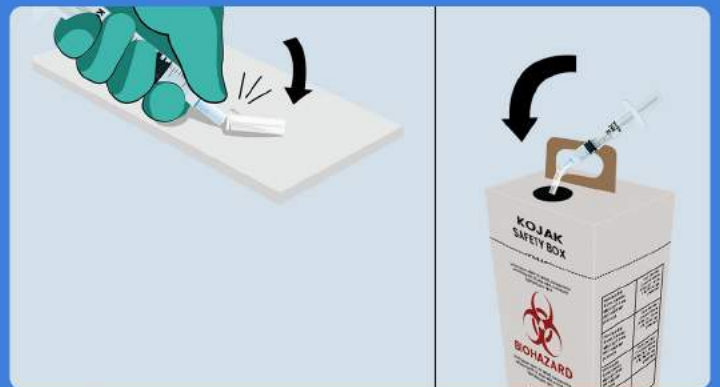

Step 6

To activate the locking mechanism, press the needle cover against a hard surface. You will hear a 'click' when the needle cover is locked. Dispose the used syringe into the sharps container.

To know more scan the QR Code





Hindustan Syringes & Medical Devices Ltd

174/25, Ballabgarh, Faridabad India- 121 004, (India)

Phone: +91-129-4289000, **Email:** info@hmdhealthcare.com

Registered Office

901, Narain Manzil, 23, Barakhamba Road New Delhi-110001(India)

Phone: 011-23314785