


DISPO VAN®

**INSULIN
PEN NEEDLES**

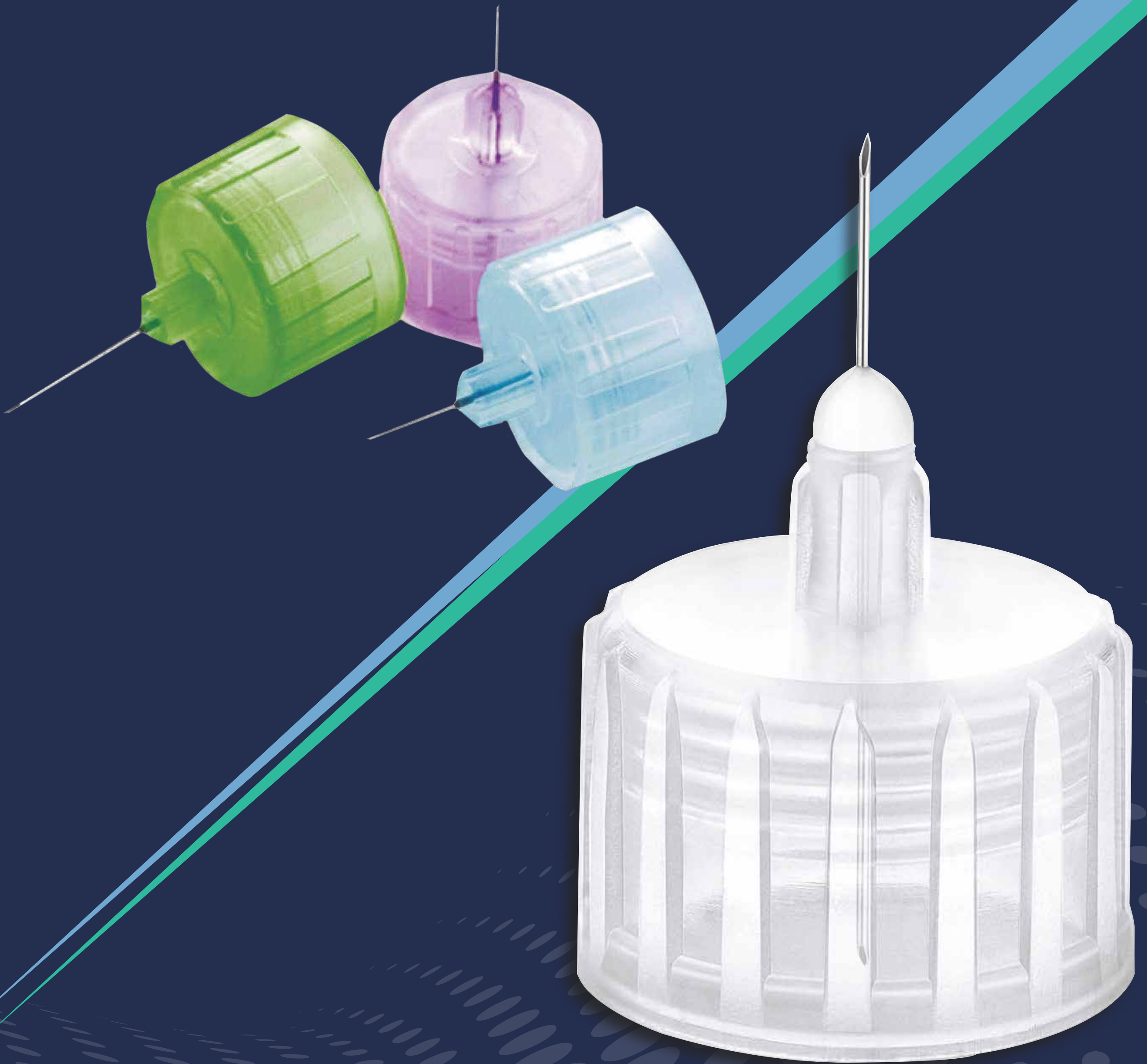

Precision Engineered, yet Affordable

MDSAP
MEDICAL DEVICE SINGLE AUDIT PROGRAM




INDUSTAN SYRINGES
& MEDICAL DEVICES LTD.
ADDRESS ISO 13485: 2016

WHY REUSE A PEN NEEDLE, WHEN **DISPO VAN**® MAKES IT AFFORDABLE FOR SINGLE USE?


Features of Precision engineered **DISPO VAN**® Pen Needle

- Extra thin wall
- Ergonomic shape for improved skin contact and grip
- Multi-bevel, tapered point
- Flat base designed to improve skin contact and comfort


How to use **DISPO VAN**® pen needle ?

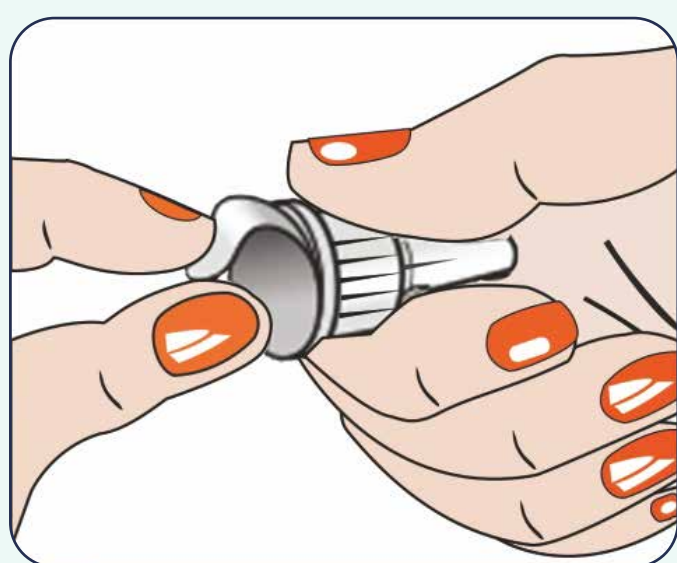


Step 1:
Take a new needle. Peel off the paper tab

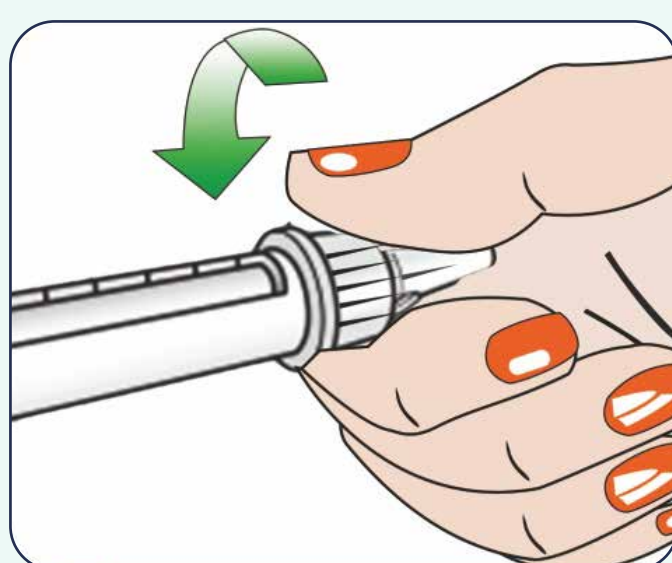

Step 2:
Push the needle straight onto the pen. Rotate until it is on tight.

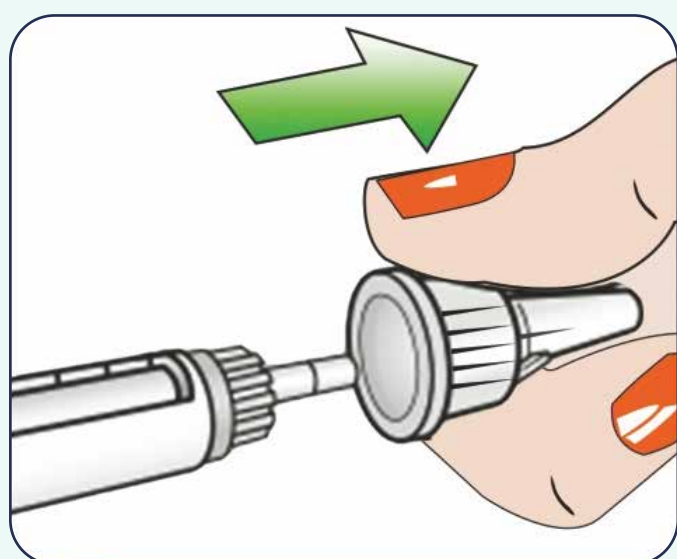

Step 3:
Pull off the outer needle cap and keep it for later. You will need it after the injection to safely remove the needle from the pen.

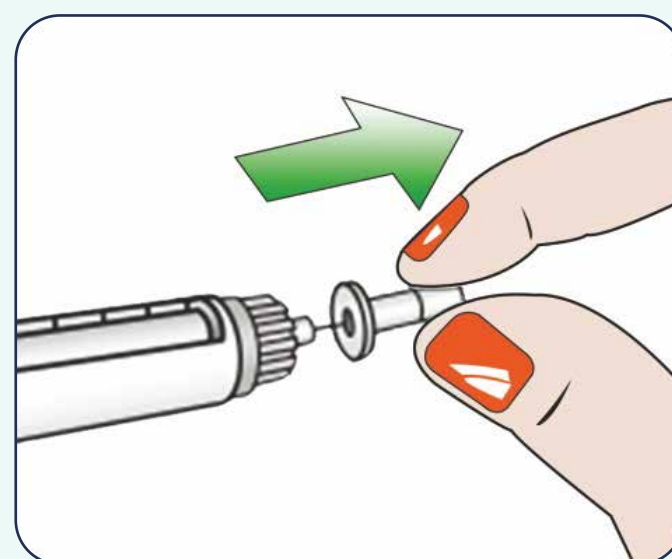

Step 4:
Pull off the inner needle cap and throw it away. If you try to put it back on, you may accidentally stick yourself with the needle. A drop of the medicine may appear at the needle tip. This is normal, but you must still check the flow.

DISPOSAL IS IMPORTANT

Discard it right away in a "sharps" container for medical waste (to prevent anyone from getting stuck with your used pen needle)

CAUTION

- Check the compatibility between the pen needle and the pen to be used for administering the medication

- Always mount a new needle before each injection and dispose of it after single use.

- Use the technique recommended by your doctor for administration.

- Store in clean, dry and insect free place.

- Avoid exposure to acidic/corrosive fume to avoid corrosion/ rust.

WARNING

- Do not use if the pack is previously opened or damaged, needles should be discard preferably in a sharp container.

- Do not attempt to re-sterilize and reuse.

REFERENCE - The "DISPO VAN" Single Use0 Pen Needle conform to the ISO 11608-2 & HSMD Specification

SHELF LIFE - The shelf life of product is 5 year from the date of manufacturing.

NEEDLE GUAGE	COLOUR CODE	NEEDLE LENGTH		NEEDLE DIAMETER
		mm	inches	
31G (Use this for heavy build patient)	PURPLE	5	3/16 [ⓧ]	0.25
32G	GREEN	4	5/32 [ⓧ]	0.23

Available in Boxes of 100 and pouches of 5



Scan to Visit Product Video



Scan to Visit Product Page