105 mm
53 mm

EN

Instruction for use :
N.B. People with reduced mental/physical capacities should be assisted while using the device. Here are the steps to follow that ensures the accurate use of the pen needle:

1. Check the expiry date of the device. Do not use if expired.
2. Check the compatibility between the pen needle and the pen to be used for administering the medication, considering the information on the packaging.
3. Always mount a new needle before each injection and dispose it afterwards. As a matter of fact, never store the pen with a mounted needle as this would leave an open path of communication between the cartridge and the outside. The pen needle is a sterile disposable device; therefore, it must be changed after each injection, otherwise you risk:
 - a) Inaccurate dosage. If the needle is left on the pen between injections, the composition of medication may be altered.
 - b) Air getting into the cartridge through the needle. The air bubbles reduce the normal flow of medication during the injection, with the consequent risk of a reduced dose of the drug.
 - c) The crystallization of the medication. If the needle is left mounted on the pen, the medication remaining inside the pen can transform into crystals, and partially or completely obstruct the needle, causing difficulty or inability to carry out the injection or a painful injection.
 - d) Contracting infections. Once used, the pen needle is no longer sterile.
 - e) Experiencing a more painful injection. During the injection, the lubricant put on the needle is completely removed and the tip of the original needle goes a progressive deformation losing its original shape. Often the tip of the needle becomes a "hook" shape, causing increased pain and possible tearing of the skin, if used again.
4. Before each injection, make sure the medication comes out properly. Always keep a spare pen needle as a replacement.
5. Use the technique recommended by your doctor for administration. Once it is inserted, press the injection button all the way down, being careful not to change the angle of the pen. If after the needle is removed from the skin, you notice medication dripping, keep the needle inserted longer next time, with the same dosage.
6. After the injection, remove the needle from the pen and dispose it in an appropriate container respecting applicable regulations and laws.

TO REDUCE PAIN Use new, short, small caliber needles and always follow your physicians instructions.

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105 mm
53 mm

105 mm
53 mm

“Dispo Van” Pen Needles fits all major Pen Devices like Novo, Lilly, Aventis

Symbols Glossary

	Caution
	Lot number/ Batch number
	Date of Manufacturing
	Used By/ Expiry Date
	Consult Instructions for use
	Non Pyrogenic
	Medical Device
	Sterile Barrier System

Symbols Glossary

	Do not Reuse
	Do not use if package is damaged
	Do not Resterilize
	Latex Free
	Keep away from Sunlight
	Keep dry
	Discard in a sharps container

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105 mm
53 mm

INSULIN DELIVERY

Manufactured By:
HMD
HINDUSTAN SYRINGES & MEDICAL DEVICES LTD.
JHARSAINTLY-JAJRU ROAD, SECTOR-59 PART-II,
BALLABGARH, FARIDABAD (HARYANA) INDIA-121004