PT

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Symbols Glossary

- Instructions for use
- Lot number/Batch number
- Date of Manufacturing
- Used By/Expiry Date
- Consult Instructions for use
- Auto - Disable Function
- Do not use if package is damaged
- Do not Resterilize
- Do not Reuse
- Keep away from Sunlight
- Do not recap the needle
- Discard in a sharps container
- Manufactured by
- Keep Dry

**STERILE EO**  
Sterilised by Ethylene Oxide

WHO PQS APPROVED

PRODUCT	PQS CODE NO.
AD syringe 0.5 ml	E008/007
AD syringe 0.5 ml	E008/039
AD syringe 0.1 ml	E008/045
AD syringe 0.3 ml	E008/079
AD syringe 0.2 ml	E008/092
AD syringe 0.25 ml	E008/095
RUP syringe 2 ml	E013/001
RUP syringe 5 ml	E013/018
RUP syringe 10 ml	E013/020
RUP syringe 20 ml	E013/039
RUP syringe 1 ml (fixed dose)	E013/056

**KOJAK®  
SELINGE**  
The Auto disable syringe

**Injection procedure**

Manufactured By :  
**HINDUSTAN SYRINGES  
& MEDICAL DEVICES LTD.**  
174, 178/25 BALLABGARH, FARIDABAD, INDIA-121004

**BREAKABLE**  
Thankfully.


JUN. 2022 / Rev.-0

**EN**

**DON'T PUSH THE PLUNGER FORWARD PRIOR TO FILLING MEDICATION.**

**Step 1:** Open the pack and take out syringe.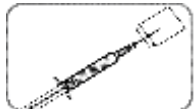


**Step 2:** Insert the needle into the vial and withdraw fluid/medicament

**Step 3:** Flick barrel to move the air bubble to top. Push the plunger for desired dosage and expel air bubble.


**Step 4:** Inject the Medicament.  
In case of 0.1 ml. AD Syringe the extended collar of the barrel facilitates limiting of the injection depth upto dermis (top layer of the skin) only & not beyond the dermis. Upon withdrawal of needle a raised wheal should be present.

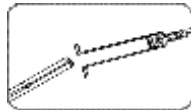**Step 5 :** Then proceed with normal injection procedure, including aspiration for enabling vein location and after giving the injection, if the plunger is pulled back, it breaks.

**Step 6:** In case of reconstitution, after step 3, pull the piston backward to (extreme) keeping the needle downward (to suck air)

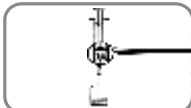**Step 7:** Release Total Fluid in Injection vial keeping the Vial downward. Then proceed with normal injection procedure by withdrawing the medicament as in step 2 followed by step no. 3, 4 & 5 \* Piston must not cross the ring otherwise A.D. Mechanism will get activated.

**Note :** Steps 6 & 7 only in case of size of all AD Syringes except 0.1 ml AD Syringes as the same cannot be used for re-constitution.

**Step 8:** Dispose off the syringe in a sharps container.

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