Sterile Single Use Safety Scalpel With Retractable Blade

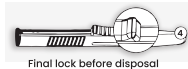
As packed

Push to take out the blade for procedure

Pull back for safety

Final lock before disposal

REPORTING OF SERIOUS ADVERSE EVENTS:

- In case of any serious adverse events, please report the same to E-mail : info@hmdhealthcare.com
- For product used in EU also report to the competent authority of the Member State in which the user and / or patient is established.

HMD NIRAJ INDUSTRIES PVT. LTD.
923, SECTOR-68, IMT, FARIDABAD-121004 HARYANA, INDIA
M.L. MFG/MD/2022/000708 Made In India

MS Manufactured under licence of Medi-Safe (Pty)Ltd (South Africa)
U.S. patent No : 8,931,181

JUN. 2024/REV.-0

English **(GB)**

INSTRUCTIONS FOR USE:

Top Activated - Retractable -Lockable - Single Use

www.hmdhealthcare.com

The reference text is the English text.

Sterile unless package is opened, damaged or broken. Intended for single use only. Do NOT resterilise.

1. Remove scalpel from sterile pack.
2. Push the slide latch forward until it clicks into position. The scalpel is now ready for use.
3. To retract the blade: Pull the slide latch back until the latch clicks into position. Visually check that the blade is fully retracted. The scalpel is now in the safety position. To use again repeat step 2.
4. Once the procedure is complete: Repeat step 3 then pull slide latch further back with force until the latch clicks into the final lock position. The scalpel is now ready for disposal.

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