“Dispo Van” Pen Needles fits all major
Pen Devices like Novo,
Lilly, Aventis

EN

You should always consult your physician regarding the most suitable needle length and injection technique to be used (at what angle, with or without pinching).

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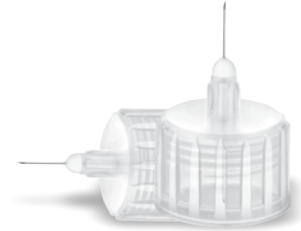
DISPO VAN[®]
PEN NEEDLE

INSULIN DELIVERY

Manufactured By:
HINDUSTAN SYRINGES & MEDICAL DEVICES LTD.
JHARSAINPLY-JAJRU ROAD, SECTOR-59 PART-II,
BALLABGARH, FARIDABAD (HARYANA) INDIA-121004
Made in India

Instruction for use :

N.B. People with reduced mental/physical capacities should be assisted while using the device. Here are the steps to follow that ensures the accurate use of the pen needle:

1. Check the expiry date of the device. Do not use if expired.
2. Check the compatibility between the pen needle and the pen to be used for administering the medication, considering the information on the packaging.
3. Always mount a new needle before each injection and dispose it afterwards. As a matter of fact, never store the pen with a mounted needle as this would leave an open path of communication between the cartridge and an outside. The pen needle is a sterile disposable device; therefore, it must be changed after each injection, otherwise you risk:
 - a) Inaccurate dosage. If the needle is left on the pen between injections, the composition of medication may be altered.
 - b) Air getting into the cartridge through the needle. The air bubbles reduce the normal flow of medication during the injection, with the consequent risk of a reduced dose of the drug.
 - c) The crystallization of the medication. If the needle is left mounted on the pen, the medication remaining inside the pen can transform into crystals, and partially or completely obstruct the needle, causing difficulty or inability to carry out the injection or a painful injection.
 - d) Contracting infections. Once used, the pen needle is no longer sterile.
 - e) Experiencing a more painful injection. During the injection, the lubricant put on the needle is completely removed and the tip of the original needle goes a progressive deformation losing its original shape. Often the tip of the needle becomes a "hook" shape, causing increased pain and possible tearing of the skin, if used again.
4. Before each injection, make sure the medication comes out properly. Always keep a spare pen needle as a replacement.
5. Use the technique recommended by your doctor for administration. Once it is inserted, press the injection button all the way down, being careful not to change the angle of the pen. If after the needle is removed from the skin, you notice medication dripping, keep the needle inserted longer next time, with the same dosage.
6. After the injection, remove the needle from the pen and dispose it in an appropriate container respecting applicable regulations and laws.

TO REDUCE PAIN Use new, short, small caliber needles and always follow your physicians instructions.

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