

# DISPO VAN<sup>®</sup>

## INSULIN PEN NEEDLES



Precision Engineered, yet Affordable



# WHY REUSE A PEN NEEDLE, WHEN **DISPO VAN**® MAKES IT AFFORDABLE FOR SINGLE USE?


## Features of Precision engineered **DISPO VAN**® Pen Needle

- Extra thin wall
- Ergonomic shape for improved skin contact and grip
- Multi-bevel, tapered point
- Flat base designed to improve skin contact and comfort


### How to use **DISPO VAN**® pen needle ?




**Step 1:**  
Take a new needle. Peel off the paper tab

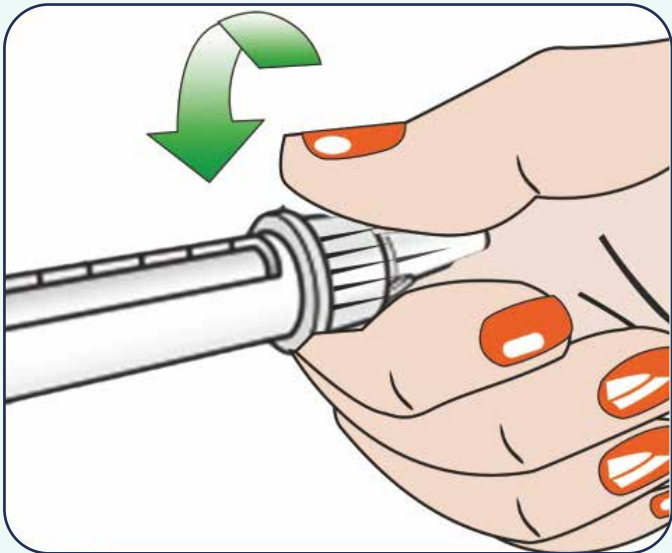

**Step 2:**  
Push the needle straight onto the pen. Rotate until it is on tight.

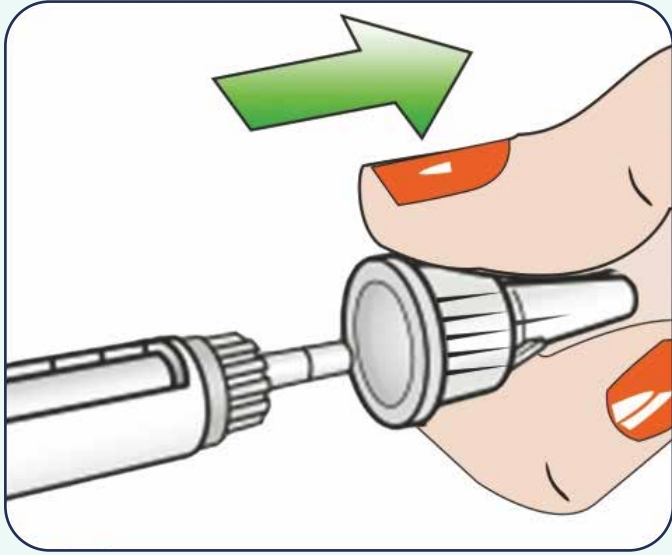

**Step 3:**  
Pull off the outer needle cap and keep it for later. You will need it after the injection to safely remove the needle from the pen.

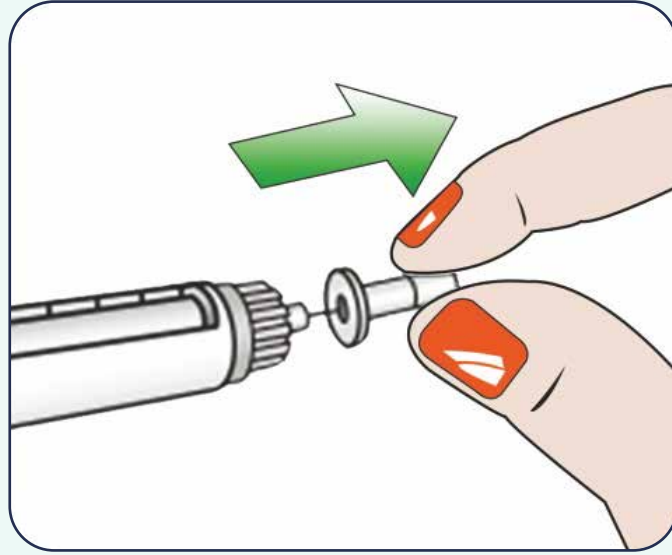

**Step 4:**  
Pull off the inner needle cap and throw it away. If you try to put it back on, you may accidentally stick yourself with the needle. A drop of the medicine may appear at the needle tip. This is normal, but you must still check the flow.

#### DISPOSAL IS IMPORTANT

Discard it right away in a "sharps" container for medical waste (to prevent anyone from getting stuck with your used pen needle)

#### CAUTION

- Check the compatibility between the pen needle and the pen to be used for administering the medication
- Always mount a new needle before each injection and dispose of it after single use.
- Use the technique recommended by your doctor for administration.
- Store in clean, dry and insect free place.
- Avoid exposure to acidic/corrosive fumes to avoid corrosion/ rust.

#### WARNING

- Do not use if the pack is previously opened or damaged.
- Do not attempt to re-sterilize and reuse.

**REFERENCE** - The "DISPO VAN" Single use Pen Needle conform to the ISO 11608-2 & HSMD Specification

**SHELF LIFE** - The shelf life of product is 5 year from the date of manufacturing.

NEEDLE GUAGE	COLOUR CODE	NEEDLE LENGTH		NEEDLE DIAMETER
31G (Use this for heavy build patient)	PURPLE	mm 5	inches 3/16"	0.25
32G	GREEN	mm 4	inches 5/32"	0.23

Available in Boxes of 100 and pouches of 5



Scan to  
Visit Product Video



Scan to  
Visit Product Page



**HINDUSTAN SYRINGES & MEDICAL DEVICES LTD.**  
174, 178/25 BALLABGARH FARIDABAD, HARYANA  
121004 INDIA

Manufacturing Site :  
**HINDUSTAN SYRINGES & MEDICAL DEVICES LTD.**  
JHARSAINLY-JAJRU ROAD SECTOR-59 PART-II  
BALLABGARH FARIDABAD, HARYANA 121004 INDIA

**Advena Ltd.**  
Tower Business Centre, 2nd Flr.,  
Tower Street, Swatar, BKR 4013 Malta