



VAKU-8 Ejecto

INSTRUCTION FOR USE (IFU)

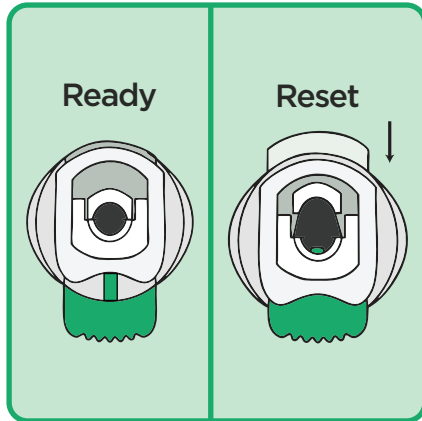


Step 1 :

(Ready / Reset) press the white reset button. Holder mechanism is ready to accept new collection device.

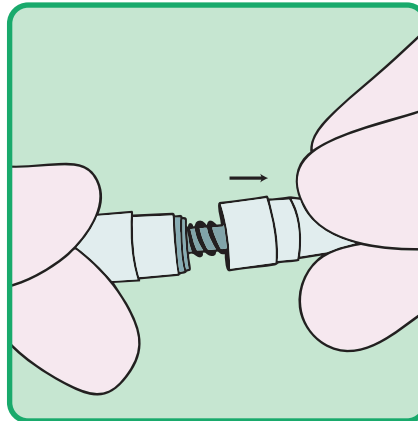

Step 2 :


Remove the non-patient end (Blood Collection Needle) BCN cover to reveal the needle, (Vaku 8 Ejecto is compatible universally with all the blood collection needles).

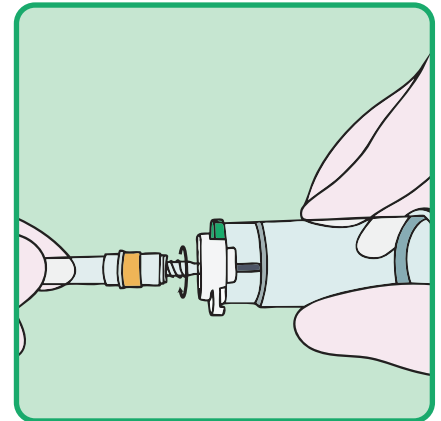

Step 3 :


Secure the BCN to the holder by screwing the hub clockwise with the holder.

Caution: DO NOT OVERTORQUE

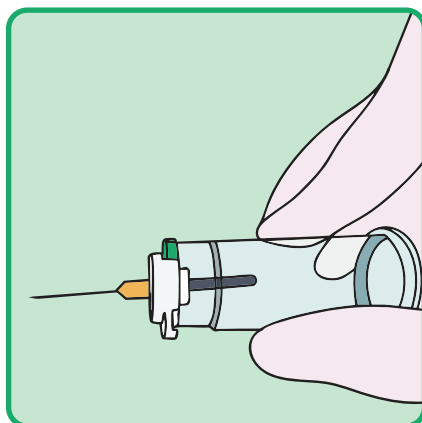


Step 4 :

Remove the patient-end needle cover and perform venipuncture according to your facility's established procedure.

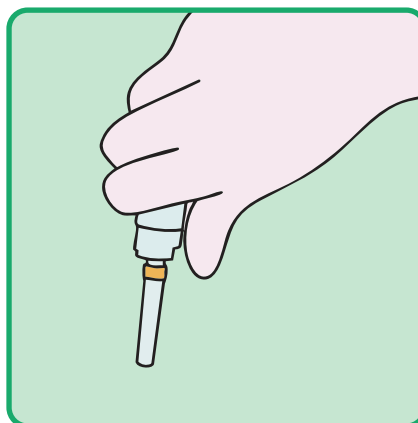

Step 5 :

Hold the device in a downward position over the sharp container opening and push the blue button to release the BCN from the holder into the container for safe disposal.

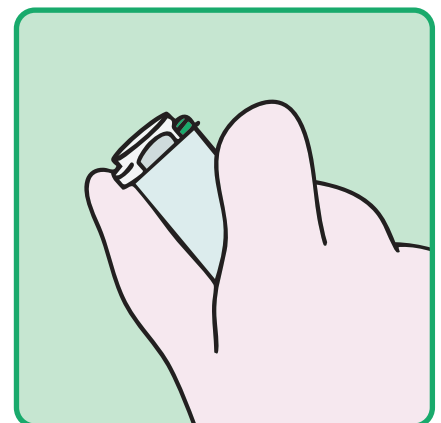

Step 6 :

Push the white button to reset and start again.



Manufacturer & Consumer Complaint Address :

HINDUSTAN SYRINGES & MEDICAL DEVICES LTD.

174, 178/25 BALLABGARH, FARIDABAD, HARYANA 121004 INDIA

CONSUMER CARE CELL NO. +91-129-4061162

E-mail : helpline@hmdhealthcare.com