

TM

DISPOCANN

ALCOHOL SWAB

For Preparation of the Skin
Prior to Injection

- Single use
- Contains 70% isopropyl alcohol
- For scrubbing, cleaning and disinfecting injection site prior to Injection
- Safer & more secure process for injection



DESCRIPTION:

Pre-Moistened Ready to Use
High Quality Anti Septic Swabs

SHELF LIFE:

Five Years

ACTIVE INGREDIENT:

Saturated with 70% v/v
Isopropyl alcohol

COMPOSITION:

Non-Woven Cellulose Swab,
Swab Size: 30 x 60 mm Isopropyl
alcohol 70%, Double distilled
water 30%, Non-Woven,
Multi-layer laminated paper

INACTIVE INGREDIENT:

Purified Water

FEATURES:

1. 70% isopropyl alcohol, contain optimal disinfectant volume to prevent bacterial growth
2. Manufactured in class 7 clean room using fully automatic machine
3. Sealed in 4 layer wrapper to ensure air tight seal to prevent drying and leakage
4. Ready to use 30X60mm swab can be used in variety of applications

INSTRUCTIONS FOR USE:

1. Wear the gloves.
2. Grasp the packet where "Tear here" is noted and tear the pouch. This will open the product so the swab is easily retrieved.
3. Do not unfold the swab.
4. Hold swab between thumb and index finger.
5. Wipe the injection site vigorously.
6. Allow the prepare area to dry.
7. Discard after use as per state or country laws and rules and regulations.

CONTRAINDICATION:

Do not apply on irritated
or hypersensitive skin,
cuts, burn, wound and on
mucous membrane.

WARNINGS: For external use only.

Do not use in eyes. Keep out of
reach of children. If swallowed,
Get medical help or contact a
poison control center right away.



HINDUSTAN SYRINGES & MEDICAL DEVICES LTD.

99/25, Ballabgarh, Faridabad (Haryana), Pincode-121004. India | www.hmdhealthcare.com | Info@hmdhealthcare.com |



Scan to
Visit Product Page

Revision No: (Jun.2025/Rev.-02)