Sterile Single Use Safety Scalpel With Retractable Blade

INSTRUCTION FOR USE

1

As packed

2

Push forward until a click is heard to expose blade for procedure

3

Pull back until a click is heard for safety

4

Final lock - Pull back with force until a loud click is heard before disposal

REPORTING OF SERIOUS ADVERSE EVENTS:

- In case of any serious adverse events, please report the same to E-mail : info@hmdhealthcare.com
- For product used in EU also report to the competent authority of the Member State in which the user and / or patient is established.

Manufacturer :
NIRAJ INDUSTRIES PVT. LTD.
923, SECTOR-68, IMT, FARIDABAD-121004 HARYANA, INDIA
M.L. MFG/MD/2022/000708

Made In India

Advena Ltd.
Tower Business Centre
2nd Flr., Tower Street Swatar, BKR 4013 Malta

JUL 2025/REV.-1

English (GB)

INSTRUCTIONS FOR USE :

Top Activated - Retractable - Lockable - Single Use

The reference text is the English text.

Sterile unless package is opened, damaged or broken. Intended for single use only. Do NOT resterilise.

- Remove scalpel from sterile pack.
- Push the slide latch forward until it clicks into position. The scalpel is now ready for use.
- To retract the blade: Pull the slide latch back until the latch clicks into position. Visually check that the blade is fully retracted. The scalpel is now in the safety position. To use again repeat step 2.
- Once the procedure is complete: Repeat step 3 then pull slide latch further back with force until the latch clicks into the final lock position. The scalpel is now ready for disposal.

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